



ProTaper Ultimate™

TIP CARD

dentsplysirona.com/protaperultimate



ProTaper Ultimate™

Shaping

Motor settings: 400 rpm / 4-5.2 Ncm

Standard




Slider 016.002v




Shaper 020.004v



F1 020.007v

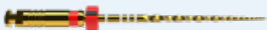

F2 025.008v

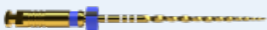

F3 030.009v

Larger and straighter canals only



Slider 016.002v



FX 035.012v




FXL 050.010v



Cleaning

Upon removing any given file, clean and irrigate with the Irrigation Needle

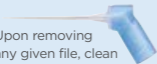


Use SmartLite Pro EndoActivator™ to remove debris



Coming soon

Obturation

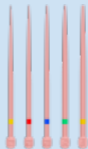

Absorbent Points
AH Plus® Bioceramic Sealer
Conform Fit® Gutta-Percha Points

Accessories



Dedicated manual version



SX
Orifice Opener

		CE
Files	Maillefer Instruments Holding Sàrl , Chemin du Verger 3, CH-1338 Ballaigues, Switzerland	2797
Gutta-Percha & Absorbent Points	Dentsply Tulsa Dental Specialties , 608 Rolling Hills Drive, Johnson City, TN 37604, USA	2797
Irrigation Needle	Produits Dentaires SA , Rue des Bosquets 18, CH-1800, Vevey, Switzerland	1639
AH Plus® Bioceramic Sealer	Maruchi , 2-208, Medical Industry Complex Bldg., 42-10, Taejanggungdan-gil, Wonju-si, Gangwon-do, KR-26311, South Korea	0197
SmartLite Pro EndoActivator™	Dentsply LLC , trading as Dentsply professional, 1301 Smile Way, York, PA 17404, USA	2797